

Safersonic

Sterile Transducer-Cover with Adhesive



- *No Rubber Bands*
- *No Artifacts*
- *No Gel at the Transducer*
- *Latexfree*
- *No Glue Remnants*

NEW: Now Intended for Intrasurgical Use!

Sterile Gel-Free-Transducer Cover with Adhesive


Adhesive eliminates the need of Gel under the Cover – no more bubbles or wrinkles.
Telescopic folding provides sterile application.

Single use Safersonic Transducer-Cover, packed in sterile pouch is now intended for intrasurgical use!

Three steps to successful use



Reach into the cover following the path of the white tab and remove the white rectangle to expose the adhesive area.

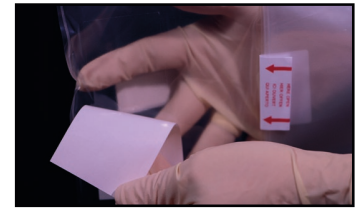

Press adhesive against the center of the Transducer using the middle-finger - to prevent wrinkles, rub/flatten adhesive area towards both ends.



The non sterile person grabs the cover and pulls it over head and cable – remove white tab and wrap around probe cover.



For IFU and videos please visit: www.safersonic.com

Item	CatNbRr	Dim	Qty
Safersonic Single	SAF. 612. 18030	18 x 30	50
Safersonic Medium	SAF. 612. 18050*	18 x 50	50
Safersonic Conti	SAF. 612. 18120	18 x 120	50
Safersonic Long	SAF. 612. 18250	18 x 250	50

* on request

DISTRIBUTOR

Safersonic Medizinprodukte Handels-GmbH

Herrengasse 5 | A - 3370 Ybbs an der Donau | Tel.: +43 (0) 7412 55772 | E-mail: info@safersonic.com